Cincha epicondilitis

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Indications:

Treatment of sport or work-related lateral epicondylitis (tennis elbow) or medial epicondylitis (golfer's elbow).


Precautions:

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop where you purchased it.
- Only a doctor is qualified to prescribe and determine the duration of treatment.
- The orthosis requires assembly and prior adjustment by qualified personnel.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- The general instructions of use and specific indications determined by the doctor should always be followed.
- Treatment should be regularly checked by qualified personnel.
- If you experience any side effects, you should immediately inform your doctor.
- Your doctor should be informed if a skin condition appears or any type of sensitisation occurs.
- Your doctor might consider the use of topical creams in conjunction with the orthosis.
- In areas of support where pressure is applied, the skin should not be damaged or hypersensitive.
- For the orthosis to function properly, its components must be regularly checked to ensure that they are in good condition.
- Do not let children play with this device.
- Clean the product regularly and maintain proper hygiene.
- The medical staff responsible for monitoring treatment may explain the suitability of the product to the patient or suggest its replacement if it is damaged or worn.
- Do not wear the orthosis while having a bath or shower, or sleeping at night.

Instructions for proper fitting:

- Before using the strap for the first time, remove the transparent film from the non-slip area.
- Fit the strap to the forearm 3 cm below the elbow.
- The blue pad on the inside of the strap should always be positioned in such a way that the two parallel pressure points are pointing upwards towards the elbow and the single pressure point is pointing downwards towards the hand (see Fig. 1).
- Tighten the strap to the required pressure and close with the Velcro.

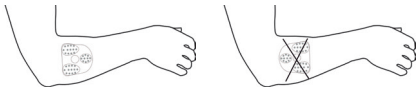Fig.1

Product maintenance - washing instructions:

Before washing the orthosis, fasten the Velcros to the strap. Hand wash. Do not use bleach. Do not tumble dry. Do not iron. Do not dry clean.

To dry the orthosis, use a towel to absorb as much moisture as possible and then leave to dry at room temperature. Do not hang up, iron or expose to direct heat sources such as stoves, heaters, radiators, direct sunlight, etc.

Note: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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PRIM, S.A