MASTER-AID®**HerpesPatch**

• • • • IT • • • •

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MASTER-AID®**HerpesPatch**

• • • • EN • • • •

Master-Aid® Cold Sore Patch is an adhesive patch with hydrocolloid which protects and treats the symptoms of cold sores. The patch protects the sore and maintains the right level of moisture to assist in the natural healing process.

Instruction for use

1. Ensure that the skin is clean and dry. Do not use the patch in combination with creams as it will not be able to stick to the skin. Wash your hands well before application.
2. Remove the patch from the package.
3. Remove the Master-Aid silicon backing paper.
4. Apply the patch onto the cold sore, being careful not to touch the adhesive part with your fingers. Use the mirror provided on the inside of the box to facilitate proper application of the patch.
5. Ensure adhesion by gently pressing around the whole surface of the patch.
6. Peel away the protective white paper.
7. Replace the patch when it starts to detach from the skin. The duration of the application varies from person to person and in relation to the condition of use (between meals or during meals).

⚠ Hygiene precautions/Warnings

1. Avoid touching the herpes-affected area.
2. If you have touched the cold sore, avoid touching your eyes and wash your hands straight away.
3. Avoid sharing glasses, cutlery, lipsticks, toothbrushes or towels with other people.
4. The patch has been designed for treating herpes labialis (cold sores). Do not use on the inside of the mouth, on mucous membranes or for genital herpes.
5. It is possible to eat and drink while wearing the patch.
6. Do not reuse the patch.

Box with mirror inside**MASTER-AID®****HerpesPatch**

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