EFFIGERM Wound Wash Liquid

RS

Effigerm Wound Wash Liquid – je rastvor za ispiranje, čišćenje i vlaženje rana. Za lečenje manjih ogrebotina, razderotina, posekotina, opekotina, ali i za primenu na neoštećenoj koži, jer pomaže u sprečavanju stvaranja biofilma.

1. Identifikacija, sastav i upotreba

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EFFIGERM Wound Wash Liquid

EN

Effigerm Wound Wash Liquid – is a Wound Irrigation Solution for cleansing, rinsing and moisturizing of wounds. For the management of minor skin abrasions, minor lacerations, minor irritations, minor cuts, minor burns and intact skin – helps prevent biofilm.

1. Identification, composition and Use

Effigerm Wound Wash Liquid is easy to use spray liquid formulation which is an electrolyzed oxidizing water solution known as hypochlorous acid that is dermatologically tested I.

For:

- Cleansing and debriding of the wound and preventing infections.
- For the management of minor skin abrasions, minor lacerations, minor irritations, minor cuts, minor burns and intact skin.
- Wound irrigation solution.
- To irrigate and moisten.
- For sore irritated skin.
- Helps with the prevention of biofilm
- Can be used for moistening the wound dressing and moisturising the wound itself as well as for loosening crusted dressings.
- Flushing of irritants associated with wound malodour.

1.1 Mechanism of action

Effigerm Wound Wash is a wound irrigation solution and is an electro chemically modified preserved water solution using hypochlorous acid It's key mode of action is to modify the wound environment by mechanical removal of foreign bodies that may cause skin irritation and infection and to provide an optimal environment with regards to moisture, humidity, oxygen and pH to assist in the healing process.

Alcohol free, suitable from birth and no need to rinse.

1.2 Components

Each bottle Effigerm Wound Wash MD contains:

- 0,016% Hypochlorous Acid
- Excipients: Ionised Water, Sodium Chloride

2. Package

The Medical Device is available in the following presentations:

60ml, 100ml, 150ml, 250ml bottles with finger spray or nozzle or 500ml bottle with trigger spray or screw cap.

3. Dosage and method for use

Apply Effigerm Wound Wash to the affected area to rinse the wound thereby removing any foreign bodies. Apply as required for optimal modification of wound environment.

4. Side Effects

In rare cases a minor sting sensation may occur.

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No known interactions.

6. Storage

Store Effigerm Wound Wash MD at room temperature maximum 25°C away from direct heat and sunlight. Do not store below 4°C. Do not transfer into another container. Recap after each application. Keep out of reach of children.

7. Warnings and precautions

Do not use Effigerm Wound Wash MD in case of known hypersensitivity or intolerance to any constituent, contact your health professional for any queries.

Do not ingest EFFIGERM Wound Wash MD.

Effigerm is suitable from birth.

Medical Device is Alcohol Free.

For best effect allow spray to dry naturally.

For Topical Use Only.

Do not use for more than 30 days.

Concomitant use of product with other medicaments should only be carried out with health professional advice.

8. Expiration Date

Do not use after expiration date printed on the packaging.

The expiry date refers to the Effigerm in its intact container when stored.

9. Disposal

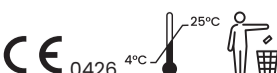
Expired device must be managed according to the waste management regulations of Local Entity Manager. Do not dispose Effigerm Wound Wash or its packaging into the environment.

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CF Pharma Ireland Ltd